


INDICAID® COVID-19 Rapid Antigen Test

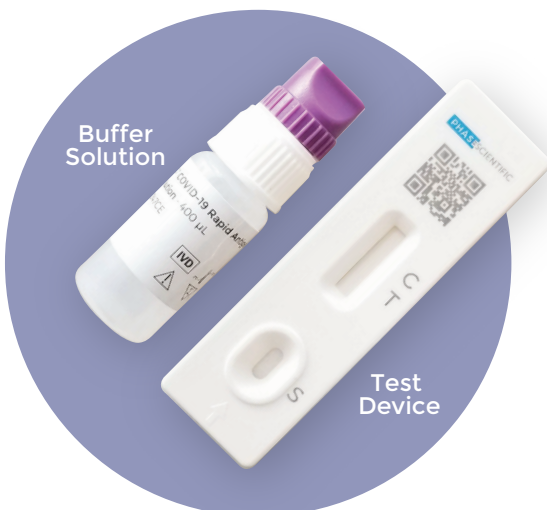

The INDICAID® COVID-19 Rapid Antigen Test is a CE-marked lateral flow immunoassay designed for the qualitative detection of SARS-CoV-2 antigens in direct nasal swab samples.



Product Advantages

- **Fast:** Intuitive visual results in 20 minutes
- **High Sensitivity:** Clinically validated with overall specificity and sensitivity at 99% and 96% respectively
- **Affordable:** Diagnosis at a fraction of the price
- **Easy-to-use:** No special equipment or facilities needed

Product Ref. No.	2110200	2120100
Package Size	25's	1's

Intuitive Visual Results



Positive Result



Negative Result

Easy-to-Use

Remove the swab and test device from their packaging.

01




Tilt your head back. Gently insert the swab about 1 inch into one of your nostrils. Twist for 5 seconds. Repeat with your other nostril.

02



The Buffer Solution Vial cap is composed of two parts. Remove the entire cap. Stir the swab into the buffer solution by twisting the swab back and forth 20 times. Ensure that the swab tip is fully submerged in the solution.

03

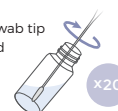


Close the entire vial cap tightly. Immediately perform Steps 5-7.

04



Remove the top half of the vial cap to expose the dropper tip.

05

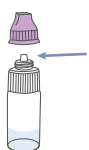

Hold the vial vertically. Squeeze and drip 3 drops of the solution into the circular opening of the test device.

06



Leave for 20 minutes and read the results. Do not read after 25 minutes. Refer to 'Interpreting Your Results' section below.

07



Find out more at www.phasescientific.com

PHASESCIENTIFIC