INDICAID® COVID-19 IgM/IgG Rapid Test

INDICAID® COVID-19 IgM/IgG Rapid Test (CE-IVD) detects anti-SARS-CoV-2 IgM/IgG antibodies in human whole blood (venous or fingerstick), serum or plasma. It can be used to assess the antibody development status after COVID-19 vaccination.

Product Advantages

- **Fast:** Intuitive visual results in 10 minutes
- **Accurate:** Independently validated with overall specificity and sensitivity at 99% and 96% respectively
- **Easy-to-Use:** No special equipment or facilities needed, anyone can use it anytime and anywhere
- **After vaccination testing:** Assessing the antibody development status after COVID-19 vaccination

Materials Supplied

- | | |
|--|--|
| <input type="checkbox"/> Test device | <input type="checkbox"/> Alcohol pad |
| <input type="checkbox"/> Safety lancet | <input type="checkbox"/> Pipette |
| <input type="checkbox"/> Buffer solution | <input type="checkbox"/> Quick Start Guide |

Test Procedure

1. Collect blood, serum, or plasma sample
2. Apply sample to the well of the test device
3. Add the buffer solution into the sample well
4. Read result after 10 minutes and no more than 15 minutes

Limitations: The test result cannot be used for diagnosis of COVID-19. If the result does not match the clinical evaluation, a confirmatory test is required.

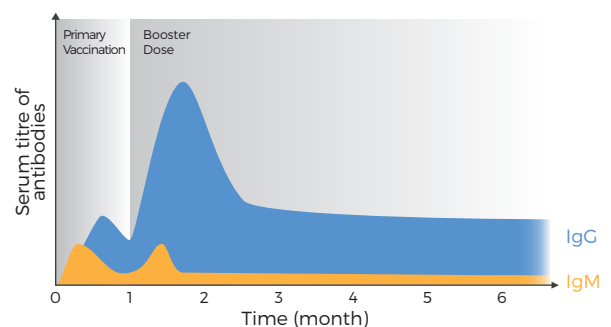
Interpreting Results

Positive	C IgG IgM	C IgG IgM	C IgG IgM	
	Presence of both IgG and IgM antibodies against SARS-CoV-2	Presence of IgM antibodies against SARS-CoV-2	Presence of IgG antibodies against SARS-CoV-2	
Negative	C IgG IgM			
No presence of IgM or IgG antibodies against SARS-CoV-2				
Invalid	C IgG IgM	C IgG IgM	C IgG IgM	C IgG IgM

Used only by medical professionals

Product Ref. No.	Package Size
2011200	1 kit / box

Vaccine Immunization Development


Suggested testing period

- 2-4 weeks **after the second dose**

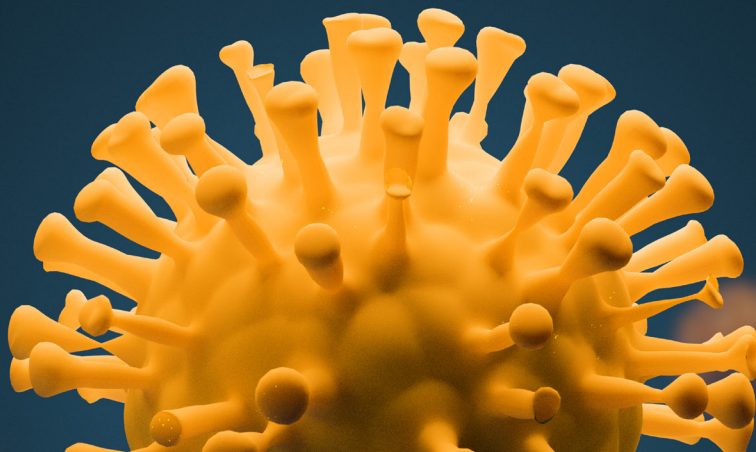Find out more at www.phasescientific.com

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