

INTENDED USE







The PHASIFY™ Sample Collection Kit (Saliva) is intended for the preservation, stabilization, and transportation of RNA collected from human saliva for up to 7 days.

The kit is for use under the guidance of medical professionals.





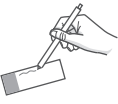


PRECAUTIONS

- Do **NOT** eat, drink, smoke, brush your teeth or chew gum before use.
- Follow guidelines as provided by your healthcare provider.

KIT CONTENTS

1 vial with an attached funnel 	1 bottle of saliva buffer 	1 vial cap 	1 patient information label 	1 specimen bag 	1 quick start guide 
---	--	---	--	---	---

HOW TO USE

-  Spit until the saliva (excluding bubbles) reaches in between the two lines. Do not overfill.
-  Add all of the saliva buffer into the saliva-filled vial.
-  Unscrew the funnel from the vial.
-  Close the tube tightly with the vial cap. Invert 10 times to mix.
-  Fill in the information on the label (or as instructed by your healthcare provider).
-  Stick the label onto the vial.
-  Place the vial into the specimen bag and seal the bag.

FREQUENTLY ASKED QUESTIONS (FAQs)

I ACCIDENTALLY SPILLED THE SALIVA BUFFER. IS IT HARMFUL?

If the saliva buffer was spilled, flush the affected area abundantly with water upon disposal. Do NOT clean saliva buffer spills with bleach.

Avoid having the saliva buffer come into contact with your eyes, skin and mouth. If contact occurs with the eyes, flush with water immediately and seek medical help. If contact occurs with your skin, wash the area with soap and rinse with water. Do not ingest the buffer. If accidental ingestion occurs, please seek medical help immediately.

WHAT DO I DO WITH THE KIT COMPONENTS AFTER COLLECTING MY SAMPLE?

Return your specimen to your healthcare provider for testing.

For remaining kit components, carefully wrap and seal all product components and dispose by throwing it into the garbage just like any household trash.

WHAT IF I HAD EATEN OR BRUSHED MY TEETH PRIOR TO COLLECTING MY SAMPLE?

Eating, drinking, smoking, brushing your teeth, or chewing gum prior to sample collection may interfere with the downstream molecular test, giving less accurate results. It is recommended to avoid eating, smoking, drinking, brushing your teeth, or chewing gum for 30 minutes prior to collecting your sample.

IMPORTANT

- All components in this kit should remain sealed before use
- Store at 15-30°C. Do not freeze. Avoid direct sunlight
- Do not use the test kit after the expiration date
- If you have questions, please contact Customer Service at +852-9135-2570 or phasify@phasesci.com.

WARNINGS

- All components in this sample collection kit are for one-time use only. Do not reuse
- Do not swallow. Avoid contact with your eyes. If contact occurs, flush with water immediately and seek medical help
- Do not clean buffer solution spills with bleach
- Vial cap may be a choking hazard. Keep away from children